	REF N5302	EN
Single Use Flow Sensor		
		STERILE R
	CE0120	
SLE Limited Twin Bridges Business Park, 232 Selsdon Road, South Croydon, Surrey, CR2 6PL United Kingdom T: +44 (0)208 681 1414 F: +44 (0)208 649 8570 E: <a href="mailto:info@sle.co.uk">info@sle.co.uk</a>		

#### Instructions for Use

The sensor is intended for use with SLE4000, SLE5000 and SLE6000 ventilators. Refer to the ventilator user manual for ventilator specific connection and calibration instructions, cautions, warnings and specifications for the sensor. The flow sensor is used to detect the movement of gasses in the breathing circuit, which enables the ventilator to display and monitor the gas flow to and from the patient enabling monitoring of ventilation status.

The sensor is a single patient use device. It is supplied sterile.

Before insertion into the patient circuit the user must calibrate the flow sensor.

The flow sensor should be calibrated every 24 hours whilst in use, if the patient's condition permits.

**CAUTION:** The flow sensor may require cleaning during use.

**CAUTION:** Do not subject the sensor to high impact loads and shocks. This may destroy the sensor wires and render the flow sensor inoperable.

**WARNING:** Do not use if the packaging has been damaged.

**WARNING:** Before use the sensor must be checked for damage. Damaged parts must not be used.

**WARNING:** The sensor cannot be re-sterilized or reused. Risk of cross contamination.


**WARNING:** Do not administer surfactant or use any nebulized gases (medications, salt solutions, etc.) in conjunction with the sensor, as they are likely to degrade the performance of the sensor and subsequent displayed accuracies.

#### Sensor Cleaning During Use

- 1-Disconnect the sensor body from the flow sensor cable.
- 2-Rinse the sensor body in sterile water with a gentle backwards and forwards motion.
- 3-Drain any water from the sensor.
- 4-Calibrate the flow sensor and re-insert sensor into patient circuit.

**WARNING:** Do not clean the flow sensor with compressed air or water jet. This may destroy the sensor wires and render the flow sensor inoperable.

	Do not use if packaging is damaged.
--	-------------------------------------

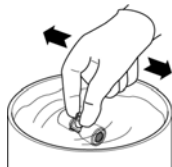	REF N5302	FR
Capteur de débit à usage unique		
		STERILE R
	CE0120	
SLE Limited Twin Bridges Business Park, 232 Selsdon Road, South Croydon, Surrey, CR2 6PL Royaume-Uni T: +44 (0)208 681 1414 F: +44 (0)208 649 8570 E: <a href="mailto:info@sle.co.uk">info@sle.co.uk</a>		

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